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Regi-trans®

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Recommendations for use **EN**

Regi-trans®

Regi-trans is a transparent matrix material based on addition curing vinyl silicones. For this purpose make an impression of the surface before beginning the preparation. This impression builds a tailor made transparent matrix for the subsequent restoration (e.g. veneers) with light curing composites.

Regi-trans is also suitable as a transparent bite registration material for fast and precise bite registration. Because of the transparency the occlusion can be visually controlled.

With the cartridge **Regi-trans** can be applied free of bubbles directly onto the teeth.

Regi-trans meets the requirements of ISO 4823, type B (bite registration materials).

Indications/Intended use

- Matrix material
- Bite registration material

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

INSTRUCTIONS FOR USE

Preparing of the cartridge

Remove and dispose the closure-cap and replace it by a supplied mixing cannula. Mount the cartridge into the application gun and extrude needed material.

Note:

Store used cartridge with fixed used mixing cannula. Do not re-use the cartridge cap. Closing the cartridge by turning on the original cap between use could cause cross contamination of components and thus premature setting.

Matrix Material

Apply the automatically mixed material onto the surface you want to restore. For better light penetration smooth the surface (e.g. place a mylar strip on the matrix). The final thickness of the matrix should be about 5 mm.

Regi-trans has a working time of 25 seconds (at 23 °C or 74 °F) from start of mixing. After a setting time of about 1:20 minutes remove the matrix from the mouth.

Prepare the tooth as usual and apply a light curing composite according to the manufacturer instructions with slight excess directly onto the tooth or into the matrix. Reposition the matrix and light cure the composite through the matrix. Remove the matrix and light cure again to guarantee complete polymerization. Remove excess material and - if necessary - check the occlusion.

If the composite (e.g. in case of deep cavities) is applied in layers, apply the final layer with slight excess, reposition the matrix and light cure.

Bite Registration

Apply the automatically mixed material directly onto the occlusal surface. Within 25 seconds after the beginning of the extruding (working time at 23 °C or 74 °F) bring the bite into the desired intercuspidation. The recommended time in mouth is about 1:20 minutes.

The transparency of Regi-Trans allows a visual control during fixing of the gypsum model in the articulator.

Disinfection

Disinfection with disinfection solutions specific for materials based on addition curing silicones is possible. Refer to the manufacturer instructions.

Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not use latex gloves because they may inhibit the polymerization of vinyl silicones. Do not touch prepared tooth with latex gloves.
- Matrixes and registrats made from Regi-Trans can be stored over months without shrinking.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Bisico, Bielefelder Dentalsilicone GmbH & Co. KG warrants this product will be free from defects in material and manufacture. Bisico, Bielefelder Dentalsilicone GmbH & Co. KGmakes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusively remedy and Bisico, Bielefelder Dentalsilicone GmbH & Co. KG ’s sole obligation shall be repair or replacement of the Bisico, Bielefelder Dentalsilicone GmbH & Co. KG product.

Limitation of Liability

Except where prohibited by law, Bisico, Bielefelder Dentalsilicone GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Supply form

REF 71900 | 3 x 50 ml cartridges | 18 green mixing tips

Made in Germany!

For dental use only!

Federal law restricts this device to sale by or on the order of a dentist

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