Best Etch

Best Etch is a 37% phosphoric acid gel for enamel and dentine with an excellent thixotropy

Delivered in syringes of 2 ml or 25 ml. With the applied very thin disposable application cannulas it can be applied very easily and safely on only that areas that are really desired to be etched.

Indications

Etching of enamel and dentine according to the enamel etch or total etch technique for:

- Composite restorations
- Sealing of fissures
- Adhesive cementation of inlays, onlays, crowns and bridges (with suitable composite-cement systems)

Warnings

Best Etch contains phosphoric acid and is corrosive. Avoid contact with eyes, mucous membrane and skin. For safety reasons it is absolutely necessary to wear gloves, safety glasses and a face mask during the filling process from the 25 ml syringe in the 3 ml syringe. During the application safety glasses are recommended for both patients and dentists. After contact with eyes immediately rinse with copious amounts of water for at least 15 minutes and consult a physician/ophthalmologist. After skin contact immediately rinse with soap and water.

Side effects

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Preparation of Best Etch syringe 25 ml

Screw off the screw caps of the 25 ml syringe and the 3 ml application syringe. Ensure that the plunger of the empty 3 ml syringe is pressed down completely. Connect the two syringes with the applied adapter with a clockwise rotation. Take care, that the two syringes are firmly connected (do not strip the thread). Store the screw caps! During the refilling, do not point the syringes at people.

Best Etch is transferred into the application syringe by slowly and steadily pulling back the plunger of the small syringe. **Never press the plunger of the 25 ml syringe!** Avoid to overfill the 3 ml syringe, never exceed the plunger safety stop due to the risk of acid burn! Disconnect the two syringes with a counter clockwise rotation. After finishing the filling process, close both syringes with the original screw cap.

Best Etch will be applied only with the application syringe.

Preparation of Best Etch syringe 2 ml or 3 ml syringe

Screw off the screw cap and substitute by the application needle. Store the screw cap!

Note:

The application needle might be bend in a slight arc, if desired, to facilitate application.

Handling precaution:

Squeeze out the first small drop on a pad to ensure proper handling and prevent "shooting" out of the etching gel by entrapped air.

Application of Best Etch

Apply **Best Etch** onto the enamel and dentine surfaces beginning with the enamel bevels. Condition **the enamel for at least 15 seconds and the dentine for 15 seconds.** (This results in 20–30 seconds etching of enamel and 15 seconds of dentine). Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water. Dry it in a water-free and oil-free airstream, but do not desiccate. The etched enamel bevel should have a chalky white appearance.

Etching precaution:

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water. After use replace disposable application cannula by the screw cap. Discard used application cannula. NeedleTips are for single use only due to hygienic reasons.

Application of Adhesive

Apply the adhesive of your choice (e.g. **Bond LC**) using a brush on the etched saliva-free enamel and dentine surfaces according to the instructions of the manufacturer.

Restorative placement, cure and finishing

Refer to manufacturers instructions for placement, curing and finishing of restorative materials.

Storage

Do not store above 25°C (77 °F). Do not use after expiry date

Warranty

Bisico GmbH warrants this product will be free from defects in material and manufacture. Bisico makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Bisico's sole obligation shall be repair or replacement of the Bisico product.

Limitation of Liability

Except where prohibited by law, Bisico GmbH, will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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bisico Bisico GmbH • Johanneswerkstraße 3 • D-33611 Bielefeld