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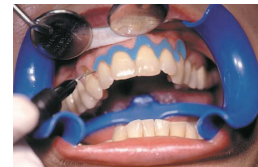
Bild 1**Bild 2****Bild 3****Bisi-Gingiva Protector****1 Intended purpose**

Light-curing material for coating/isolating the gingiva from the teeth to be treated during in-office bleaching

2 Description of product and users

Bisi-Gingiva Protector is a light-curing gingiva protector and is used, like a rubber dam, to isolate the gingiva from the teeth to be treated in dental practice.s.

2.1 Patient target group

All patients who are undergoing tooth whitening with highly concentrated whitening gel in a dental practice. The Bisi-Gingiva Protector is used as a protective barrier and is applied prior to tooth whitening.

2.2 Users

For use in the dental practice by dental professionals

3 Composition

Aliphatic urethane dimethacrylate, tetramethylene dimethacrylate, glass powder sil., light initiators, stabilisers, pigment

4 Indications

For coating gingiva during in-office bleaching procedures

5 Contraindications

If a patient has allergies or hypersensitivities to a component of this product, it should not be used or used only under the strict supervision of the attending physician/dentist.

6 Warnings

Contains aliphatic urethane dimethacrylate, tetramethylene dimethacrylate. May cause an allergic skin reaction.

7 Safety instructions

Avoid breathing vapours/spray. Wear protective gloves. If skin irritation or rash occurs: get medical advice/ medical attention.

8 Interactions with other medicinal products

The dentist should consider known interactions and cross-reactions of the medical product with other materials already in the patient's mouth before using the product.

9 Application / Preparation**9.1 Preparatory work**

Use suitable measures (such as cheek/lip retractor) to ensure that tongue and lip are fixed during treatment (and during the subsequent bleaching procedure) and do not come into contact with the material. Remove the cap on the syringe and replace it with the top of the syringe. Check that the material flows freely.

9.2 Application

Dry the gingiva. Spread a layer of Bisi-Gingiva Protector approximately 4 - 6 mm wide and 2 mm deep along the area of application. (Photo 1)

Press material into the interdental spaces. Cure with a standard handheld polymerisation lamp. Polymerise for 20 to 40 sec, ensuring that the lamp does not stay in one place for too long. Carefully remove any surplus material left on the teeth after curing with a scalpel. The bleaching gel can now be applied. (Photo 2)

9.3 Completion

Remove Gingiva Protector when the bleaching procedure is completed. Use an instrument (e.g. a probe) to lift the material up at one end and draw it off. Remove the top of the syringe and replace it with the cap. (Photo 3)

10 Storage and handling

Close the device tightly after use. Store at 3°C - 25°C.

11 Shelf life

The maximum shelf life is printed on the label of each syringe. Do not use after the expiry date.

12 Side effects

With proper preparation and use of this medical device, adverse effects are extremely rare. However, immune reactions (such as allergies) or local discomfort cannot in principle be ruled out completely. All serious incidents which occur in connection with the use of this product are to be reported to the manufacturer indicated below and the competent authority in each case..

13 Instructions for disposal

Leftover quantities and packaging materials are to be disposed of according to the local and/or statutory regulations.

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Rev.: 700763 v. 20251016

Bisi-After-Care**1 Intended use**

Bisi_After-Care is a paste to be applied on the teeth for post-treatment after a bleaching procedure.

2 Product description and user**2.1 Product specific description**

Bisi-After-Care is a water-based paste that reduces tooth sensitivity after teeth have been bleached.

2.2 Patient target group

To be used on all patients whose teeth have just been bleached.

2.3 User

The Bisi-After-Care is used by the dentist in the dental practice directly on the patient after a bleaching treatment.

3 Composition

Purified Water, Xylitol, Glycerin, Potassium Nitrate, Sodium Fluoride, Thickener, Flavourings

4 Indications

For after-treatment of the teeth after a bleaching procedure.

5. Contraindications

If the patient is allergic or hypersensitive to one of the components, this product must not be used or only under the strict supervision of the treating doctor/dentist.

6 Interactions with other agents

Well-known cross-reactions or interactions of the medical device with other materials already in the mouth must be taken into account by the dentist when using it.

7 Application / Processing

1. If necessary, dab excess saliva from tooth surfaces.
2. Apply a generous amount of Bisi-After-Care to the tooth surface using a disposable brush, glove or, in difficult interproximal situations, an interproximal brush.
3. Leave Bisi-After-Care in the patient's mouth for approx. 3 to 5 minutes.
4. Ask the patient to rinse their mouth well. The patient should not eat or drink for at least 30 minutes after the treatment.

8 Notes on storage and handling

Store at 3 - 25 °C.

9 Shelf life

The maximum shelf life is printed on the label of each syringe. Do not use after the expiration date.

10 Side effects

Undesirable side effects of this medical product are extremely rare if processed and used properly. In principle, however, immune reactions (e.g. allergies) or local discomfort cannot be completely ruled out. All serious incidents occurring in connection with the use of this product must be reported to the manufacturer specified below and to the relevant competent authority.

11 Instructions for disposal

Residual quantities and packaging material must be disposed of in accordance with local and/or statutory regulations.

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Bisico Bielefelder Dental silicone GmbH & Co.KG
MANI MEDICAL GERMANY GmbH