COMPO-JECT™

Applicator

Operating Instructions
Gebruiksaanwijzing
Mode d'emploi
Gebrauchsanleitung
Brugsvejledning

IMPORTANT NOTE:

COMPO-JECT™
IS DESIGNED FOR
PRE-FILLED TIPS ONLY



COMPO-JECT™ is a high pressure device for the direct application of medium-high viscosity composites in pre-loaded unit dose tips. The Clip-in-lock which is convenient and time saving is not suitable for use with self-filled tips as leakage may impair the moving parts of the device.

OPERATING INSTRUCTIONS

GB

COMPO-JECT™

Applicator for dental materials in prefilled unit-dose tips for direct oral application.

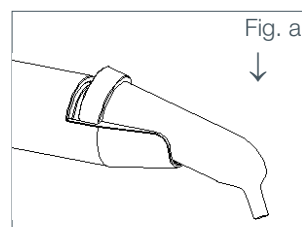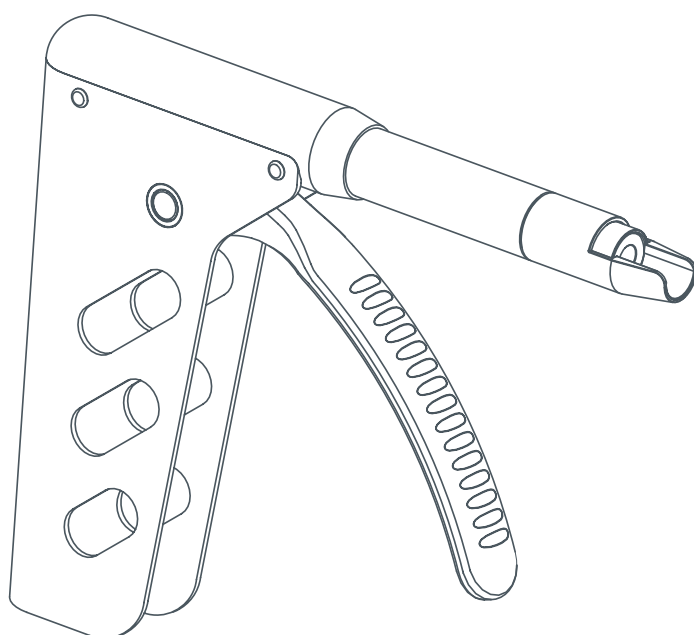- Insert an unit-dose tip into the clip-in-lock at the front part of the applicator. Make sure that the tip is fully inserted.
- For easy placement of the material, the tip can be rotated in the required direction.
- Activate the lever at all times while using the applicator in the oral cavity. This will cause the piston to lock the unit-dose tip and thus prevent unintentional rotation or release of the tip.
- Release the tip from the clip-in-lock by pressing the front part of the tip downwards (Fig. a).

Cleaning and Sterilization

COMPO-JECT™ can be cleaned and sterilized by any method up to 180°C/356°F.

Materials

COMPO-JECT™ is made from high grade, acid-resistant stainless steel.

Guarantee

COMPO-JECT™ is covered by a 5-year material and construction guarantee. Damages due to inadequate care or incorrect handling are not covered by the warranty.

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RØNVIG Dental Mfg. A/S, Gl. Vejlevej 59, DK-8721 Daugaard, Dänemark.
RØNVIG Dental Mfg. A/S, Gl. Vejlevej 59, DK-8721 Daugaard, Danmark.