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Recommendations for use **EN**

## Regidur i®

**Addition curing extremely hard and fast setting vinyl impression silicone for bite registration**

With **Regidur i®**, you have an extremely hard and quick-hardening silicone at your disposal which meets the daily requirements in all respects.

**Regidur i®** can also be used for sealings of total prosthetic registrations, e.g. when making intraoral pressure-dependent registrations or registrations by means of a face bow.

**Instructions for use**

Insert the cartridge in the dispenser. Remove the sealing cap. Put the mixing tip onto the cartridge and turn it by 90° until it locks in position. **Regidur i®** is now ready to be applied in the patient’s mouth, preferably to the lower dental arch. Whereas the patient closes the mouth, check if correct dental occlusion is ensured.

By means of a **BRT** (Bite-Registration-Tip), Regidur i can be applied to the closed row of teeth from the outer position to the left or to the right. Particularly in case of abnormal occlusion conditions like for example a cross bite, this feature can be quite advantageous.

After a period of about 45 seconds, the material has reached its final hardness and can be removed from the mouth. Excessive material can be removed with a knife and trimmed with a milling cutter or a stone.

**Important information**

Only very rarely, allergies or other intolerances may occur. In this case, **Regidur i®** should not be used!

Before first use, insert the new cartridge into the dispenser, remove the sealing cap (do not put the sealing cap onto the cartridge again!) and squeeze out a small quantity of the material until the material comes out of both openings. Only now insert the mixing tip and apply **Regidur i®**.

The mixing tip used remains on the cartridge until the next application.

Only just before the next application, the mixing tip used is removed and replaced by a new one!

Polysulfide impression materials, ferruginous(III) retraction fibres and solutions as well as constituents of latex gloves may impair the reaction of **Regidur i®**. To avoid this, the areas concerned should be rinsed thoroughly. (This list makes no claim to be complete.)

Do not use **Regidur i®** for general dental impression purposes. Its final hardness may cause problems when removing the impression from the mouth.

Do not use **Regidur i®** after date of expiry!

Please follow the instructions for use of the dispenser!

**Disinfection**

Bite registrations made of **Regidur i®** can be treated with all disinfectants generally used in the dental area (e.g. 2 % glutaraldehyde solution).

**Storage**

The material can be stored at room temperature (20 - 23°C). Higher storage temperatures reduce the reaction time, lower temperatures prolong it. Maximum storage temperature:<25°

**Technical data**

Linear shrinkage rate:	max.:	0,1%
Final hardness:	approx.	90 shore-A
Mixing time:		not applicable!
Processing time (from beginning of mixing):		15 seconds
Hardening time in the mouth (from beginning of mixing)	approx.	45 seconds

##### Packaging

<b>Regidur i®</b> in cartridges:	
3 x 50 ml cartridges (incl. 18 mixing tips pink)	<b>REF:</b> 71570
10 x 50 ml cartridges (incl. 50 mixing tips pink)	<b>REF:</b> 71575

<b>Accessories:</b>	
Mixing tip pink, 50 pieces	<b>REF:</b> 01970
Mixing tip pink, 100 pieces	<b>REF:</b> 01975
BRT (Bite-Registration-Tip), 50 pieces	<b>REF:</b> 02100
BRT (Bite-Registration-Tip), 100 pieces	<b>REF:</b> 02105

##### Federal law restricts this device to sale by or on the order of a dentist.

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Bisico • Bielefelder Dentalsilicone GmbH & Co. KG • Johanneswerkstraße 3 • D-33611 Bielefeld