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Cementime DC

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Instructions for Use

Cementime DC

Semi-permanent Cement for Implantation Purposes

Product description

Cementime DC is a dual cure semi-permanent luting cement. Cementime DC is based on multifunctional methacrylic esters and contains zinc oxide.

Cementime DC is suitable for semi-permanent cementing of implant abutment retained crowns and bridges. It shows good elastic behavior and good seal thanks controlled slight expansion. Cementime DC is delivered in 4:1 syringes so it is very easy and comfortable to use.

Cementime DC does not influence the setting of resin-based filling and luting materials.

Indications/Intended use

Cementing of crowns and bridges on implants

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Application

Preparing the Syringe

Remove the cap of the syringe and throw it away (do not use it again!). It is replaced by a special 1:1 mixing cannula. Turn the cannula 90° until it locks in position. The material is now ready for application.

Note:

Store used syringe with fixed used mixing cannula. Discard the first 2-3 mm (about the size of a peppercorn) of the extruded material. This has to be done for each new mix.

Cementing

Before cementing degrease, clean (e.g. with alcohol) and dry abutment and restoration.

Cementime DC is automatically mixed when dispensed with slight and even pressure. It can be applied directly into the restoration.

Apply a thin layer of Cementime DC to the inside of the restoration. Position the restoration within 2 minutes (working time at 23°C) cautiously but firmly over the implant abutments.

For a fast fixation of the restoration cure Cementime DC at the marginal regions for 20 seconds per surface. In case of translucent materials it is also possible to expose to light through the material (40 seconds per surface).

Without any light Cementime DC sets in the mouth within approx. 4 minutes.

After Cementime has set remove any excess.

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Notes

If the restoration has to be removed for inspection, this must be done gently with slight and permanent traction. Avoid jerky removal! The remaining cement can be extracted almost in one piece. Use water spray and suction to remove any remaining residue. Clean and disinfect abutment and restoration with a cotton pellet soaked in alcohol or another disinfectant. Rinse thoroughly with water and dry. Now the restoration can be fixed again with Cementime DC (see Cementing).

Storage

Do not store above 20 °C (68 °F)! Protect from direct sunlight. Do not use after expiry date.

Additional Notes/Warnings

- Contact of resin pastes with skin should be avoided, especially by anyone having known allergies to methacrylates.
- In case of contact with skin and eyes rinse immediately with plenty of water.
- If swallowed seek medical advice.
- Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Composition

Dimethacrylates, zinc-containing fillers, silicon dioxide, catalysts, photo initiators

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed>).

Warranty

Bisico GmbH warrants this product will be free from defects in material and manufacture. Bisico GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Bisico GmbH's sole obligation shall be repair or replacement of the Bisico GmbH product.

Limitation of Liability

Except where prohibited by law, Bisico GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.