**Gingiform / Gingiform soft**

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Rev.: 65920-03.2018

**Gingiform / Gingiform soft**

**Gingiform** is a new addition curing Vinyl silicone for the producing of elastic gingival masks by implant works as well as in the high quality metal and ceramic technique.

Gingival masks of **Bisico Gingival** provide valuable information for the perfect structuring of the working models gingival areas.

- No mixing and dosing problems, because the material will be worked out bubbles free from automatic mixing cartridges.
- Exact tension free reproduction of the gingival areas of the implant post and prepared teeth.
- Formstable within months and durable repairable by highest dimension stability and a shrinkage 0,1%.
- The final hardness of 70 shore A makes possible to work with a scalpel, a scissor or a sand cap.
- The enclosed separating spray assures the perfect separation between the Gingiform and the impression silicone.

**Picture 1**

The impression of the implant post will be dammed with Lutefix and treated with the Gingiform separator.

**Picture 4**

With this the gingival mask is finished and the model can be made.

**Picture 2**

The dammed area with the posts will be filled with **Gingiform**. With the aid of the IOT you can fill it from below under avoidance of occluded gas.

**Picture 5**

The impression prepared with the gingival mask will be poured out with model gypsum and set in the already prefilled base former.

**Picture 3**

After hardening of the silicone material, the dam will be removed and the borders of the gingival mask will be trimmed with a scalpel.

**Picture 6**

The working model of an implant work with an elastic and repairable gingival mask.

**Gingiform** is available in the following package sizes:

REF		
05400	Gingiform, 70 Shore A	1 cartridge, 50ml, 6 mixing tips, 6 IOT, spray separator 85 ml
05410	Gingiform, 70 Shore A	1 cartridge, 50ml, 6 mixing tips, 6 IOT
05411	Gingiform soft, 40 shore A	1 cartridge, 50ml, 6 mixing tips, 6 IOT
05420	Gingiform Separator	Separator 85 ml: Silicon/silicon

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