



Bisi-White Office

Bisi-White Office - a complete and effective in-office teeth whitening solution for dental professionals. This advanced system offers everything needed for professional-grade teeth bleaching, delivering impressive results for your patients.



Effective In-Office Bleaching

1 Complete Product Set

Bisi-White Office system includes all necessary components for a full in-office whitening treatment.

2 Gingiva Protection

A Dental Dam (Gingiva Protector) is included to safeguard sensitive gum tissue during the procedure.

3 Easy Application

The Automix-System Bisi-White ensures easy and precise gel application. No mixing necessary.

4 Post-Treatment Care

After-Bleaching Care products are provided to maintain results and protect teeth.





Bisi-White Whitening Gel



Hydrogen Peroxide Base

Bisi-White uses a hydrogen peroxide formula for effective whitening.



35% Max Concentration

The gel contains a powerful 35% concentration for optimal results.



5g Automix Syringe

Conveniently packed in a 5g automix syringe for easy application.



4:1 Gel to Activator Ratio

Perfectly balanced with 4 parts gel and 1 part activator.

Bisi-White offers simple, direct application with no mixing necessary, saving time and ensuring consistent results.

Bisi-White Benefits

Effective Whitening

Bisi-White provides powerful whitening for discolored or devitalized teeth, delivering long-lasting results that patients will love.

Optimal Formula

With a neutral pH value and ideal consistency, Bisi-White is gentle on teeth while still being highly effective.

Visual Contrast

The red color of Bisi-White gel provides a perfect contrast to the tooth, allowing for precise application and easy removal.

No lamp

The whitening process works without light or heat. So no lamp or laser is necessary.

If there is any bleaching gel left over after the treatment, it can simply be stored in the refrigerator for the next patient/treatment.

High Temperature Transport Solution

Temperature Concerns

High temperatures during transport can be a concern for whitening products.

Innovative Solution

Bisi-White Office addresses this with a unique "safety clamp" on the syringe plunger.

Chemical Stability

This safety feature prevents unwanted chemical reactions during shipping and storage.

Quality Assurance

Ensures the product maintains its effectiveness until it reaches your dental office.





Gingiva Protector

1

Light-Curing Composite

The Gingiva Protector is a blue light-curing composite that provides excellent contrast to the gums.

2

Quick Application

It cures with LED, plasma, or halogen lamps in just 20 seconds, saving valuable time compared to traditional rubber dams.

3

Optimal Properties

With its ideal viscosity and low heat generation, it offers comfortable protection for patients during the whitening process.



After-Bleaching Care

1

Natural Remineralization

The After-Bleaching-Care cream promotes natural remineralization of teeth after whitening treatments.

2

Pleasant Experience

Patients will enjoy the refreshing peppermint flavor, making post-treatment care more enjoyable.

3

Versatile Use

Suitable for use after both In-Office and Home bleaching procedures.

4

Protective Ingredients

Contains xylitol, potassium nitrate, and fluoride to rebalance minerals and protect against sensitivities and dental caries.

Bleaching Process

1



Gingiva Protector

Applying the dental dam and polymerise for 20 to 40 sec.

2



Bleaching Gel

Applying the Bisi-White Bleaching gel and remove it after 10 min. If necessary, repeat x 2 (total: 3 cycles)

3



After-Bleaching - Care

Apply After-Bleaching-Care to remineralise teeth.



Price

Set for 1 patient (incl. Gingiva Protector, Whitening Gel,
After-Bleaching-Care and mixing tips) **29,- €**

Refill Bleaching Gel, 4 syringes **59,- €**

Refill Gingiva Protector, 4 syringes **20,- €**

Refill After Care, 4 syringes **20,- €**

**Order now your Bisi-White
test set**