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BisiSEAL MTA

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Instructions for Use

BisiSEAL MTA

Ready-to-use bioactive permanent root canal sealer based on MTA

Product description

BisiSEAL MTA is a ready-to-use bioactive root canal sealer based on MTA for permanent root canal fillings that can be applied directly into the root canal. **BisiSEAL MTA** has a high pH value of 12-13 and is used in combination with gutta-percha.

BisiSEAL MTA is an insoluble, radiopaque one component material which sets due to the physiological liquid present in the root canal.

Indications/Intended use

- Permanent sealing of the root canal in combination with gutta-percha

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Sealer Do not use **BisiSEAL MTA** in patients with a known allergy to any of the components of the product. An allergic reaction may require re-treatment.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

Working time

No mixing is required.

BisiSEAL MTA can be applied immediately and introduced directly into the root canal. The working time at room temperature is about **1 hour**.

Setting Time:

Setting time is about 4-5 hours. However, in very dry root canals, the setting time can be more than 10 hours.

The setting time of **BisiSEAL MTA** depends on the presence of physiological liquid in the dentin, which is naturally present in the dentin. Therefore, it is not necessary to add moisture in the root canal prior to placing the material.

Root canal sealing

Use of rubber dam is recommended

Prepare and clean the root canal.

Dry the canal using paper points.

Screw off the cap of the syringe and attach one the supplied

EndoTips to the syringe by turning it clockwise.

After use of the syringe replace disposable EndoTip by the screw cap.

EndoTips are for single use only due to hygienic reasons.

Inject **BisiSEAL MTA** passively into the root canal, taking into account the previously determined working length. Filling should occur from bottom upwards by pulling the syringe tip slowly out of the root canal. Insert gutta percha into the canal up to the working length.

Using a heating device to cut the remaining excess of the gutta percha at the coronal part. Condense the gutta-percha for a better sealing with a plugger.

Seal the cavity tightly.

Additional remarks

- Store **BisiSEAL MTA** in the original aluminium bag.
- Keep away from children!

Composition

MTA, radiopaque mineral oxide

Storage

Store at 10-20 °C (50-77 °F) in the original aluminium bag and protected from moisture. After each use place the syringe back in the aluminium bag and seal it well. Use within 6 months after first opening. Do not use after expiry date.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed>).

Warranty

Bisico GmbH warrants this product will be free from defects in material and manufacture. Bisico GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Bisico GmbH's sole obligation shall be repair or replacement of the Bisico GmbH product.

Limitation of Liability

Except where prohibited by law, Bisico GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.