**Best Metal Primer****Metall-Kunststoff-Verbundsystem****Anwendung**

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**Best Metal Primer****Metal-acrylic bonding system****Application**

**Best Metal Primer** is used as a bonder between acrylics / composites and all types of metal.

After preparation, the surface to be faced is sandblasted with aluminium oxide (50 µm, 2 bar) and cleaned with a steam cleaner or acetone. A thin layer of Sebond is applied with a brush immediately after cleaning and allowed to dry for approx. 1 min. A second layer can be applied if required.

**Storage**

Sebond should be stored at 10 -25 °C. Close the bottle properly immediately after use and protect against the effects of direct light.

**Shelf life**

The maximum shelf life is printed on the label of each bottle. Do not use after the expiry date.

Each bottle contains: 5 ml

**Guarantee**

Our technical instructions, regardless of whether they are provided verbally, in writing or during practical demonstrations, are based on our own experience and should only be considered guidelines. As our products are subject to continued development, we reserve the right to modify them. dialog complies with EN ISO 10477.

**Note**

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

**Side-effects**

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted sideeffects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

**Contra-indications / interactions**

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.

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Bisico • Bielefelder Dentsilicone GmbH & Co. KG • Johanneswerkstraße 3 • D-33611 Bielefeld