Bond LC

Light cure one component bonding agent for the total-etch-technique

Bond LC is a simple to use light cure one component bonding agent. It is designed for strong bonding of light cure composites and compomers to etched enamel and dentine and to nonprecious and precious metals.

Its strong adhesion to etched enamel or dentin proceeds on principles similar to that occurring with glassionomer cements. Good, long lasting adhesive strength and good biocompatibility are attained by polycarboxylic acids.

Bond LC is compatible with all current brands of visible light cure methacrylate based composite materials. It is ethanol-based and hydrophilic. **Bond LC** can be used on slightly moist dentin surfaces (wet bonding technique).

If a dual or self cure priming and bonding system is recommended **Bond LC** can get dual cure when it is applied as a 1:1-mixture of **Bond LC** and **Activator**. The activator is available separately. For these applications refer to recommendations for use of **Activator**.

Side effects

In singular cases, **Bond LC** may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.

Bonding of Light Cure Composite and Compomer Filling Materials**1. Isolation**

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (0.5 - 1.0 mm) bevel placed in the enamel to increase the surface area for greater bond strength.

3. Pulp Protection

Cavity floor of deep excavations should be covered with a thin layer of calciumhydroxide material.

4. Enamel and Dentine Conditioning

Recommended is the total etch technique. Apply Best Etch onto the enamel and dentine surfaces beginning with the enamel bevels. Condition the enamel for at least 15 seconds and the dentine for 15 seconds. Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water. Dry it in a water-free and oil-free airstream, but do not desiccate.

A slightly wet dentine surface is important for the function of Bond LC. The etched enamel bevel should have a chalky white appearance

Etching precaution:

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

For the typical indications of compomers etching is mostly not recommended, but it improves considerably the bonding with tooth material.

5. Application of Bond LC

Apply **Bond LC** generously with a brush onto the enamel and dentin surfaces for 30 seconds with agitation. The material should build a homogeneous layer. Remove excess material carefully. Dry cautiously with oil free air for about 15 seconds to remove all volatile components and to disperse the adhesive to an even layer. Do not desiccate the dentine.

Cure the **Bond LC** coating by exposing its entire area to a dental halogen light unit for 20 seconds before application of a second layer of **Bond LC**.

Notes: Do not rinse off the **Bond LC**! If not used immediately, place dispensed **Bond LC** in subdued light to prevent premature polymerization by incident light. The **Bond LC** will not self cure.

6. Application of a second Layer of Bond LC

Apply again the **Bond LC** generously with a brush onto the adhesive surfaces as described under point 5. before placement of a light cure composite.

7. Restorative Placement

Apply the restorative material according to the instructions of the manufacturer.

Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

Storage of Bond LC

Do not store above 25 °C (77 °F)! Avoid storage in direct sunlight. Do not use after expiration date.

Warranty

Bisico GmbH warrants this product will be free from defects in material and manufacture. Bisico makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Bisico's sole obligation shall be repair or replacement of the Bisico product.

Limitation of Liability

Except where prohibited by law, Bisico GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!**For dental use only!****Delivery Form**

Bond LC

Bottle 5 ml, 50 brushes,
1 brush holder, 4 palettes (6-x)

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