Cementime DC (mini syringe)

Semi-permanent Cement for Implantation Purposes

Cementime DC is a dual cure semi-permanent luting cement. **Cementime DC** is based on multifunctional methacrylic esters and contains zinc oxide.

Cementime DC is suitable for semi-permanent cementing of implant abutment retained crowns and bridges. It shows good elastic behavior and good seal thanks controlled slight expansion. **Cementime DC** is delivered in 4:1 mini syringe so it is very easy and comfortable to use.

Cementime DC does not influence the setting of resin-based filling and luting materials.

APPLICATION**Preparing the mini syringe**

Remove the cap of the mini syringe and throw it away (do not use it again!). It is replaced by a 4:1 mixing cannula. Turn the cannula 90° until it locks in position. The material is now ready for application.

Note:

Store used syringe with fixed used mixing tip. Discard the first 2-3 mm of the extruded material. This must be done for each new mix.

Cementing

Before cementing degrease, clean (e.g. with alcohol) and dry abutment and restoration.

Cementime DC is automatically mixed when dispensed with slight and even pressure. It can be applied directly into the restoration.

Apply a thin layer of **Cementime DC** to the inside of the restoration. Position the restoration within 2 minutes (working time at 23°C) cautiously but firmly over the implant abutments. For a fast fixation of the restoration cure **Cementime DC** at the marginal regions for 20 seconds per surface. In case of translucent materials it is also possible to expose to light through the material (40 seconds per surface).

Without any light **Cementime DC** sets in the mouth within approx. 4 minutes.

After **Cementime DC** has set remove any excess.

Notes:

Removing the crown and bridge for control purposes is simple and can be made completely without instrument.

The typically used instrument (i.e. "crook") will not work because of the sticky-elastic condition of **Cementime DC**.

With the aid of a piece of tissue or cellulose, the object will be powerful picked with the fingers, dissolved and removed with permanent extraction and rhythmic movements.

If the restoration has to be removed for inspection purposes, this must be done gently with slight and permanent traction. Avoid jerky removal! **Cementime DC** can be extracted very easy and almost in one piece. Use water spray and suction to remove any remaining residue, if there are still some at the abutment or on the restoration.

Finally clean and disinfect abutment and restoration with a cotton pellet soaked in alcohol or another disinfectant. Rinse thoroughly with water and dry.

Now the restoration can be fixed again with **Cementime DC** (see cementing).

Precautions

- Do not use **Cementime DC** in cases where known allergies to the components exist.
- Contact of resin pastes with skin should be avoided, especially by anyone having known resin allergies.
- In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary.
- If swallowed seek medical advice.
- Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of **Cementime DC**.
- Neither store the material in proximity of eugenol containing products, nor let it allow coming into contact with materials containing eugenol.

Storage

Do not store above 20 °C! Do not use after expiry date.

Warranty

Bisico GmbH warrants this product will be free from defects in material and manufacture. Bisico makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Bisico's sole obligation shall be repair or replacement of the Bisico product.

Limitation of Liability

Except where prohibited by law, Bisico will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children!
For dental use only!****Delivery form**

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