Algi-Press

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Algi-Press

Vinyl Polysiloxane Impression Material ISO 4823

Algi-Press is a medium-viscosity, addition-curing silicone impression material, which combines the advantages of additioncuring silicones with those of alginates and can be used in place of typical alginate applications.

Areas of application:

- Anatomical impressions
- Opposing jaw impressions
- Production of temporary restorations
- Production of splints
- Orthodontic applications

Instructions for use:

Preparation of the cartridge

The use of **Algi-Press** requires a conventional mixing and dispensing machine. Please observe the instructions for use of these devices closely (on request, Bisico delivers the "Sympress" unit from Renfert, which is generally available on the market). Hold the cartridge with the sealed outlets facing upwards and remove the cap by grasping the tab and lifting upwards. Avoid exerting excessive force when doing so.

Insert the cartridge into the output device as per the manufacturer's

instructions and place a mixing tip over the outlets of the cartridge. Once the mixing tip is correctly positioned (the internal hexagon of the mixing tip must be correctly aligned with the hexagon of the drive shaft), push the yellow bayonet ring over the mixing tip as far as it will go and turn it clockwise (1/4 turn) to lock it securely in place.

Ensure that the drive hexagon engages in the mixing tip when the machine starts. The mixing spiral in the mixing tip should rotate when the machine is started. The tray can now be filled.

Notes:

When using a cartridge for the first time, discard the first 2-3 cm of material to adjust the levels of the two material components. Leave the used mixing tip on the cartridge as a cap until the next application.

To remove the used mixing tip, release the bayonet ring by turning it anticlockwise and then pull off the mixing tip.

Information on use:

Algi-Press should be processed at room temperature (approx. 23 °C). Higher temperatures shorten while lower temperatures extend the processing and setting time. The cartridge should be checked for damage before initial use. Defective cartridges must not be used.

The use of products containing active sulphur or nitrogen compounds

and retraction cords containing ferric (III) sulphate prior to impression taking may inhibit proper setting.

Also avoid contact with latex gloves.

Impressions with **Algi-Press** can be cast 30 minutes after removal from the mouth. They can be cast several times and also stored for extended periods of time without loss of dimensional stability. Prevent the product from coming into contact with clothing, as it cannot be removed by dry cleaning.

Do not use in combination with other impression materials.

Side effects

No side effects are known to date. Local hypersensitivity reactions to components of **Algi-Press** cannot be ruled out in individual cases. If swallowed, seek medical advice in case of discomfort or other complaints. Should the product accidentally come into contact with the eyes, rinse thoroughly with water immediately and seek medical advice if complaints persist.

Disinfection

The impressions can be disinfected with a 2 % glutaraldehyde solution or disinfection solutions, for example. Please follow the manufacturer's instructions.

Bonding agent

We recommend the use of Bisico Adhesive. Coat the impression tray with a thin layer, allow to dry for 1 minute and then cover the tray with a sufficient layer of **Algi-Press**.

Storage

Do not store at temperatures above 25 °C (78 °F)! Do not use after the expiry date!

Impressions made with **Algi-Press** can be stored for several months without any shrinkage.

Disposal

The plastic cartridge must be completely emptied before disposal and can be disposed of via a system designed for plastic waste. Uncured material is always treated as hazardous waste, whereas cured material (e.g., impressions) can be disposed of as domestic waste.

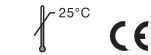
The cartridge is composed of pure, separable plastics and is therefore completely recyclable, as no composite materials (e.g., metal/plastic films) are used.

Technical data

Mixing ratio:	5:1
Consistency:	Medium
Mixing time:	N/A
Processing time:	1:20 minutes
Intraoral curing time:	01:40 minutes
Final hardness:	Approx. 45 shore A
Linear shrinkage:	Max. 0.15 % (24 h)
Elastic recovery:	98 %
Deformation under pressure:	5 %

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bisico® Bielefelder Dentsilsilicone GmbH & Co.KG • Johanneswerkstraße 3 • D-33611 Bielefeld